starr/rigid
115mm

German package

elastisch/flexible
200mm

100 Stk./pcs.

Export package

SILVERIN® STICKS

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SILVERIN® STICKS

SILVER NITRATE – POTASSIUM NITRATE CAUSTIC STICKS

MEDICAL DEVICE

- FOR THE REMOVAL OF GRANULATION TISSUE
- FOR THE REMOVAL OF UMBILICAL GRANULOMA
- FOR APPLICATION IN THE FIELDS OF GYNAECOLOGY
- FOR WOUND MANAGEMENT
- FOR STOMA CARE
- FOR THE TREATMENT OF WARTS AND CORNS

Silver Nitrate Sticks are a device containing Silver Nitrate and Potassium Nitrate on its tip. They have been mentioned in pharmacopoeias throughout history and are applied topically for the chemical cauterisation in various fields. Silver Nitrate has a bactericidal, astringent, caustic effect. When moistened, it leads to a denaturation (coagulation) of the proteins and subsequent dying off of the treated tissue. The purpose of using Potassium Nitrate is to mitigate the effect of Silver Nitrate. Silverin® Sticks allow precise and selective application in order to confine treatment to the specific areas.

Therapeutic indication:

Silver Nitrate Sticks remove excess granulation tissue which can develop in healing wounds of the skin, e.g. surgical cavities with granulation tissue. They are suitable for the use in wound management, stoma care and for the treatment of aphthous stomatitis. Silver Nitrate Sticks are used for haemostasis of smaller bleedings such as nosebleeds (epistaxis) or e.g. cervical nicks after biopsies or where electrocautery would be inconvenient and painful. They are often used for the removal of warts and in the fields of podology (e.g. for the removal of corns). Residual tissue of umbilical cords of new born babies can easily be removed,

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the treatment should be carried out by a physician or midwife. The application is quick, simple and mostly does not cause any pain. For the treatment different concentrations (50%, 75%), lengths (115mm - 4.5", 200mm - 7.9") and materials (wooden sticks – rigid, plastic sticks – elastic) are available. Various groups of healthcare professionals have formed their own preferences and there are no formal recommendations regarding strength, length or material.

Instructions for use:

With the exception of the treatment of warts Silver Nitrate Sticks must only be used by healthcare professionals. The tip of the stick should be moistened with water. If there is enough moisture on the area to be treated, it may not be necessary to moisten the stick. The aim is to transfer the water of the tip, rich in Silver Nitrate, to the area being treated. The area should be touched by dabbing and rolling the tip several times, without rubbing or pressing, taking care to use exact precision. The treated area will discolor and shed off from the surrounding tissue after a few days. It may take up to 24 hours until the treated area turns black. If necessary the treatment can be repeated ever 2–3 days until the desired results have been achieved. Silver Nitrate Sticks should be only applied on the area to be treated and are intended for single use only.

Warnings and precautions:

- The contact of Silver Nitrate Sticks on newborns and on delicate skin should be short in duration. For all unusually coloured or unusually shaped skin changes that cannot be clearly identified as granulation tissue or normal warts, the Silverin® Sticks are only intended for use under medical supervision.
- Clothing and other objects should be protected when handling the sticks. Silver Nitrate causes brown-black stains which cannot be removed with conventional cleaning agents.
- Healthy skin that has been unintentionally touched by the tip of the stick must be thoroughly cleaned using running water and a sponge; despite cleaning, contact with the tip of the stick can result in a harmless dark colouration of the affected areas of the skin. The dark colour fades after about 10–14 days.
- If the Silver Nitrate Stick comes into contact with the eye, it should be rinsed immediately under running water for at least 5 minutes with the eyelid open and then a doctor should be consulted.
- When treating mouth and throat, it is important to ensure that no parts of the Silver Nitrate Sticks can be swallowed, as this can lead to serious health problems.

Content and pack sizes:

Composition:

Silver Nitrate and Potassium Nitrate

Available concentrations:

50% Silver Nitrate
75% Silver Nitrate

Pack sizes:

10 and 100 pieces, length 115 mm (4.5")
10 and 100 pieces, length 200 mm (7.9")

Both sizes are available as rigid and flexible sticks.